



MUSICC Identification System (Multi-Use Sterilizable Integrated Chip Carrier)

AVID Injector

AVID3001

The AVID injector is a Monoject syringe with a removable needle assembly. Designed for subcutaneous and intramuscular injections of the AVID microchip into various animals, reptile and fish. The needle assembly may be reused for more than one microchip injection. The injector

Now Available in 10mm Avid or ISO Code!

is also designed to work with the Multi Use Sterilizable Integrated Chip Carrier (MUSICC) storage container. For best results, it is recommended that 70% Isopropyl Alcohol be used when cold sterilization is performed.



MUSICC Identification Systems Available in:

- 12mm - AVID2023 (AVID Code)
- AVID2123 (FECAVA)
- AVID2323 (ISO)
- 10mm - AVID20231 (AVID Code)
- AVID23231 (ISO)

The AVID microchip dispenser (MUSICC) holds 25 AVID microchips. The microchip dispenser is economically designed for portability and convenience. The 25 AVID microchips, in the microchip dispenser, are packaged and sealed in a plastic vial. The microchip dispenser can be stored in the vial and submerged in an accepted cold sterilization solution. By rotating the top of the dispenser (see the Top View), the microchip can be removed through one of two openings in the top on the dispenser. The microchip is removed with the AVID3001 injector: the injector needle can be stored in the microchip dispenser with the recommended cold sterile solution.



12mm MUSICC - Available in AVID, Fecava, and ISO Code!
10mm MUSICC - Available in ISO or AVID Code!

Top View



Call Now Toll-Free **(800) 336-2843** Sales Option 3
<http://AvidID.com>